

FM-101 Community Facemask

Not a medical mask. No FFP standards met. No certification.



shop.ant-neuro.com

Manufacturer: eemagine Medical Imaging Solutions GmbH
Gubener Str. 47, D-10243 Berlin
Germany

email: info@eemagine.com

Description: The FM-101 is a reusable face mask (community mask, courtesy mask) covering mouth and nose. It is designed to help reduce the velocity of respiration flow or expelled bodily fluids (coughing or sneezing), to aid in the awareness of social distancing measures and to contribute towards the reduction of the further spread of SARS-CoV-2.

See also (in German)

<https://www.bfarm.de/SharedDocs/Risikoinformationen/Medizinprodukte/DE/schutzmasken.html>

Construction: The fabric part of the mask consists of 3 layers: the inner and outer layers are made of *CoolMax Elastic*[®] polyester/elastane fabric, a quick-drying, highly breathable textile that is used in sports and medical products. The invisible intermediate layer consists of 0.5 mm thin *Polibond*[®] non-woven fabric made of polypropylene fibers. This functional fleece layer has an air permeability of 155 l / min according to ISO 9237 (specification from the manufacturer).

Important: The FM-101 face mask is not a medical product and did not pass any regulatory approval that may qualify it as a medical product or item for personal protection. Please keep this product information safe for the duration of use (do not store it in the same bag as the used mask).

The FM-101 face mask should not be used on young children under the age of two, or people who have difficulty breathing or are passed out, are unable to act, or are otherwise unable to remove the mask without help. In extreme cases, the cord contained in the mask can lead to strangulation. Store the masks so that they are not accessible to small children.

For further and up-to-date information about the product, please visit our web shop: shop.ant-neuro.com. Here you will also find information on the available sizes.

Application: Please wash before wearing for the first time! When putting on a mask, please make sure that the inside is not contaminated. The hands should be washed thoroughly with soap before manipulating the mask. The mask must be properly placed over the mouth, nose and cheeks and fit as closely as possible around the edges to minimize air penetration on the sides. The wire piece in the fabric at the level of the nose can be bent accordingly. The mask is attached to a cord with the two transparent spacers behind the ears and tightened with the clip at the back of the head if necessary.

Potentially pathogenic compounds may accumulate on the outside of the used mask: To avoid contamination of the hands, touching it should be avoided if possible. After removing the mask, the hands should be washed thoroughly in accordance with the general hygiene rules (at least 30 seconds with soap).

Storage and washing: After removal, the mask should be kept airtight in the packaging bag or washed immediately. It should only be kept unwashed for as short amount of time as possible. The masks can be washed at 60 °C and should afterwards be dried completely. Under no circumstances may the mask be ironed! Hot air drying (hair dryer) can significantly reduce the lifetime of the mask.

The mask is designed to sustain its shape and functionality after approx. 150 washing cycles (including centrifuging). The mask should not be tumble dried.

Disposal: The mask can be disposed of with normal household waste.

Guarantee: The manufacturer accepts no liability for any risk of infection and/or transmission of contagious material.

Complaints and feedback: Please write us an email to info@eemagine.com or contact us via our web shop at shop.ant-neuro.com.